

3M Science.
Applied to Life.™

3M™ True Definition Scanner

3M™ Margin Marking Software Installation Quick Start Guide

3M Margin Marking Software requires:

- 3M™ Connection Center Account and Login
- 4GB RAM and Microsoft Windows 7 or later on your computer system

Installation of 3M Margin Marking Software

1. Download and save the 3M_Margin_Marking_Software.exe from <https://connectioncenter.3m.com/vamm-updates/>
2. Once the file is downloaded, right click on the file and select **Run as administrator** (Figure 1).

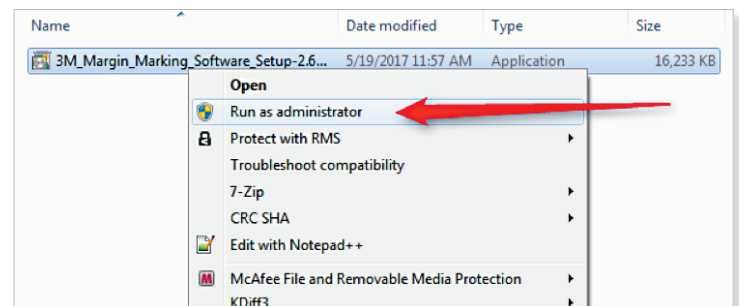


Figure 1

Installation of 3M™ Margin Marking Software (cont.)

3. If you receive an install error, proceed to the following steps:
 - a. For Windows 7, install the following update by clicking on the link: <http://www.microsoft.com/en-us/download/details.aspx?displaylang=en&id=26347>
 - i. Click **download** and on next screen select **vcredist_86.EXE**, then click **Next** (Figure 2)
 - ii. **run**, and the install will begin
 - b. For Windows 10, proceed to the following steps:
 - i. Open the **Start Menu**
 - ii. Type “**windows features**”
 - iii. Select **settings** on the search pane
 - iv. Click on **Turn Windows Features On and Off**
 - v. Find and click on **.Net Framework 3.5** (includes .NET 2.0 and 3.0)
 - vi. Complete the install wizard
4. Repeat step 2.
5. Open the 3M Margin Marking Software, by clicking on the icon (Figure 3).
6. Login with your 3M™ Connection Center username and password.
 - a. If you get the following error when logging in (Figure 4), check the username/password used.

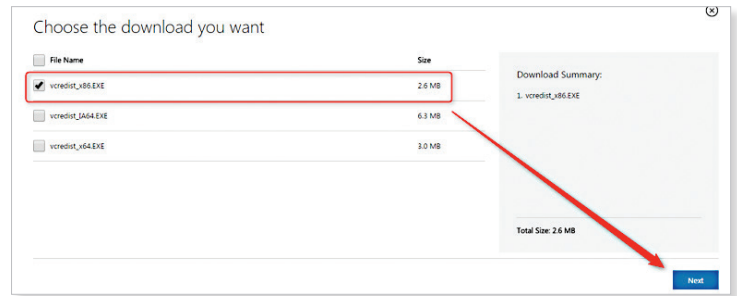


Figure 2



Figure 3

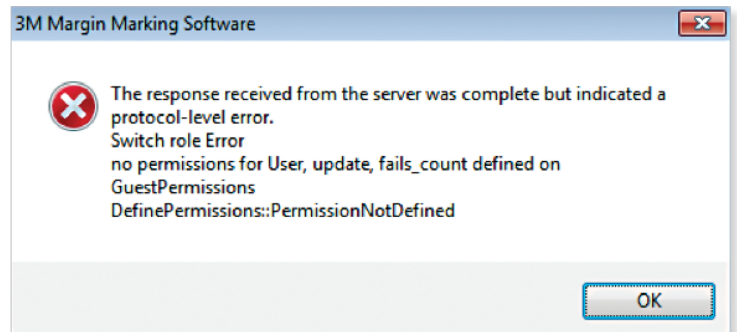



Figure 4

 **NOTE:** To utilize the 3M Margin Marking Software, the user must go through the 3M Margin Marking certification process to have full capability of the software.

3M Customer Care Center: 1-800-634-2249
3M.com/TrueDef



3M Oral Care
2510 Conway Avenue
St. Paul, MN 55144-1000 USA

Phone 1-800-634-2249
Web 3M.com/TrueDef

3M Canada
Post Office Box 5757
London, Ontario N6A 4T1
Canada
1-888-363-3685

3M is a trademark of 3M. Used under license in Canada.
Please recycle. Printed in U.S.A. All other trademarks are the property of their respective owners. © 3M 2017. All rights reserved. 70-2013-0944-3